



GORILLA®/BABY GORILLA® PLATE TEMPLATES TECHNIQUE _____

Paragon 28®, Inc. offers various sterile packaged implants and instrumentation. Plate Templates are available for use with sterile packaged Gorilla®/Baby Gorilla® Plates.


The Plate Templates are intended to be used as trials to help identify the appropriate sterile packaged Gorilla®/Baby Gorilla® Plate for use within the surgical site. To assist with handling and placement of certain Plate Templates, a raised boss is located within the Plate Template where a provided Precision® Guide Set Screw can be attached. The Plate Templates can be temporarily secured to the surgical site with olive wires, if desired.

When using the Templates in an extracutaneous manner, it is important to consider the varying thickness of surrounding soft tissues. Under fluoroscopy, identify the fracture/joint line in relation to the Plate Template. The location of Plate screw holes are indicated by the holes along the Plate Template. All Plate Templates are intended to help identify the appropriate sterile packaged Gorilla®/Baby Gorilla® Plate size or configuration. Select Plate Templates may have laser markings to help identify the lengths of sterile packaged Gorilla®/Baby Gorilla® Plates.

Based on the surgeons' assessment of the anatomy and using the Plate Template as a guide, the appropriate Gorilla®/Baby Gorilla® Plate is identified and presented to the sterile field using an aseptic transfer technique. Refer to P99-STR-1001 for Aseptic Transfer Techniques of Paragon 28 sterile packaged configurations. For the contraindications, potential complications and adverse reactions, warnings and precautions associated with this device, please refer to the device specific instructions for use at <http://www.paragon28.com/ifus>

P51-STT-1001 Rev A

™Trademarks and ®Registered Marks of Paragon 28®, Inc.
© Copyright 2021 Paragon 28®, Inc. All rights reserved.
Patents: www.paragon28.com/patents

Paragon 28, Inc. 
14445 Grasslands Dr.
Englewood, CO 80112 USA
(855) 786-2828

Paragon 28 Medical Devices Trading Limited
First Floor Block 7 Beckett Way
Park West Business Park
Dublin 12
D12 X884
Ireland
+353 (0) 1588 0350



www.Paragon28.com