

BONE MARROWTM
ASPIRATE KIT

Bone Marrow Aspirate Kit



Bone Marrow Aspirate Kit

Bone Marrow Harvest Needle Design

- ▶ Small 11 gauge needle designed to help minimize soft tissue disruption during retrieval of marrow at harvest site
- ▶ Easy insertion using a double diamond tip design for a faster harvest time




The Bone Marrow Aspirate kit is a great ancillary product to the PRESERVE™ Precut Allograft Wedge System

The Bone Marrow Aspirate kit is a great ancillary product to the PRESERVE™ Precut Allograft Wedge System

Lure Lock Syringe

Lure Lock syringe secures itself to the Harvest Needle construct to draw Bone Marrow continuously in an airtight environment

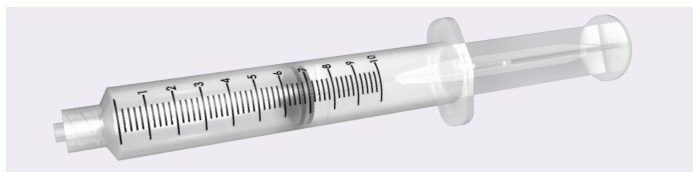


Allograft Features


- ▶ Aseptically processed—no gamma irradiation or bleach
- ▶ Density matched
- ▶ Available for Evans, Cotton, MTP, Subtalar, Calcaneal Cuboid and Lapidus procedures



PRESERVE SMO Wedge

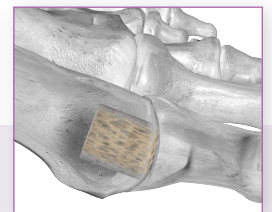


PRESERVE AVITRAC™

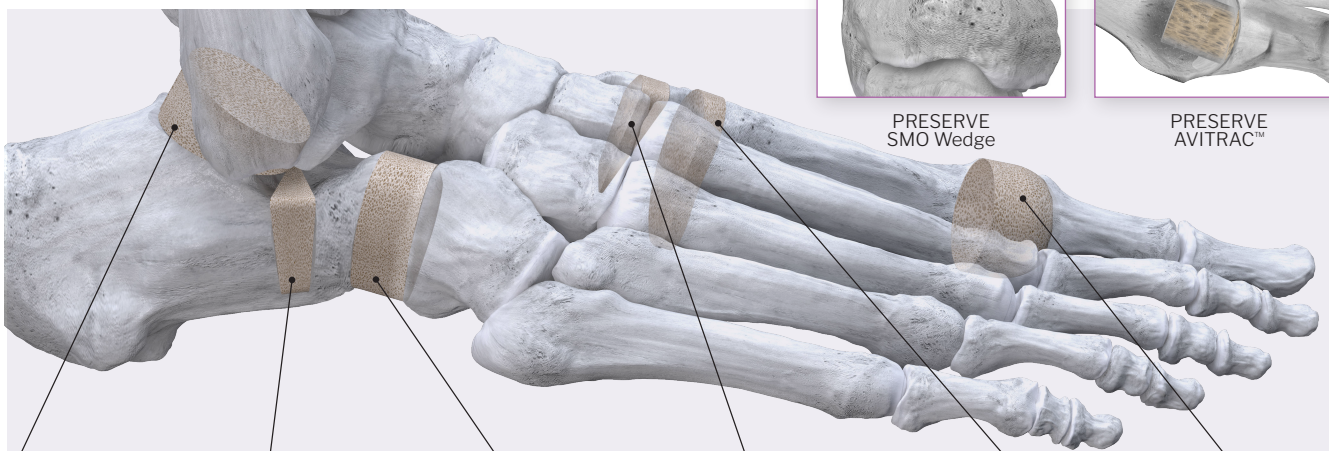

PRESERVE Subtalar Graft




PRESERVE Evans Graft

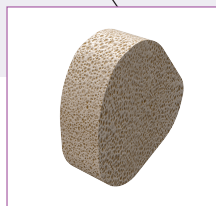


PRESERVE Calc-Cuboid Graft



PRESERVE Cotton Graft

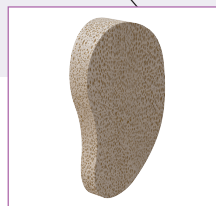

PRESERVE Lapidus Graft

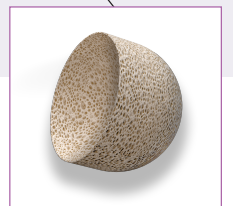


PRESERVE MTP Graft


SURGICAL TECHNIQUE




Make small incision at chosen harvest site. Insert needle into harvest site. Advance through cortex into marrow.

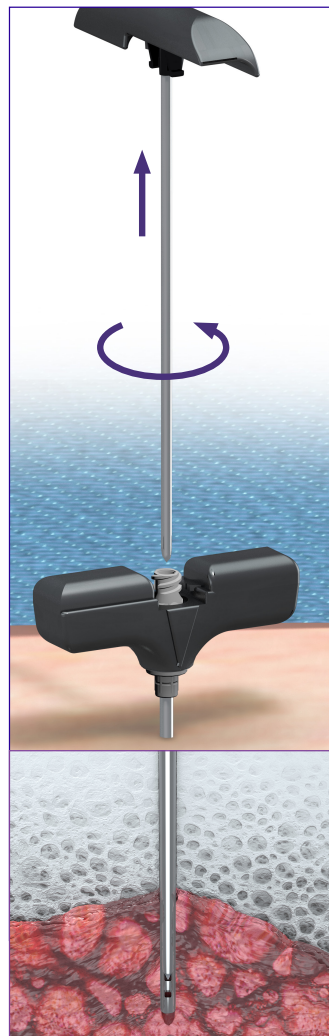

Twist top of handle 90° and remove needle from outer cannula.

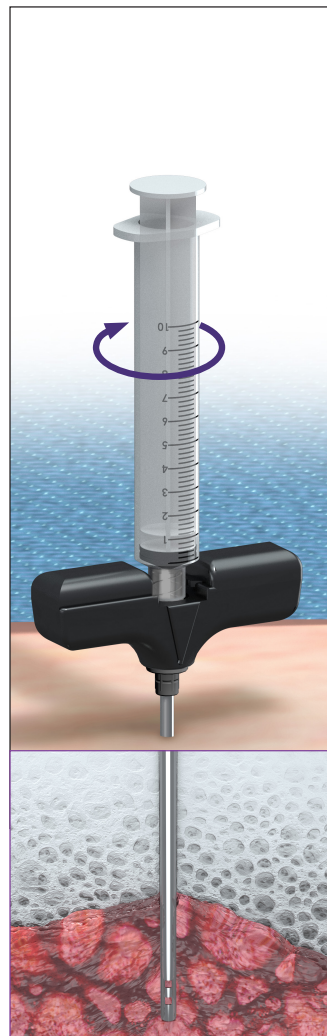

Secure lure lock syringe onto handle base.

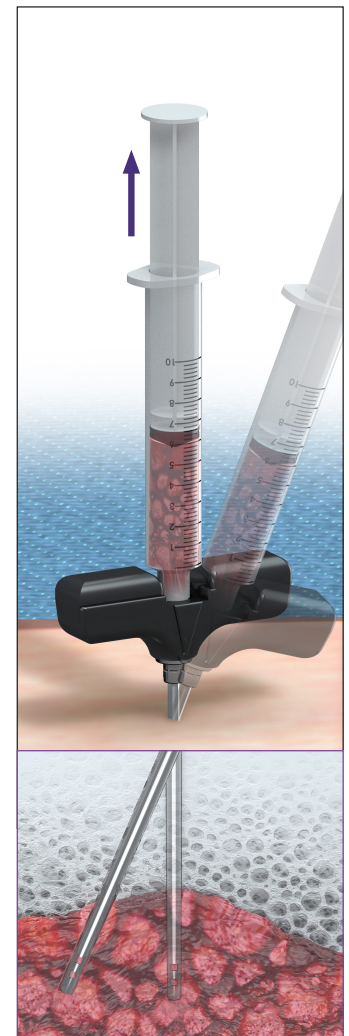

Extract bone marrow and re-position as needed to maximize bone marrow pull.

Bone Marrow Aspirate Kit Contents


Description	Quantity
Bone Marrow Harvest Needle, 100 mm, 11 Gauge, Sterile	1*
Syringe, Locking, 10 cc, Sterile	1*

*When ordered in the Bone Marrow Aspirate Kit, five individually packaged harvest needles and syringes will be provided; however, the facility will only be billed for the quantity used.

BONE MARROW
ASPIRATE KIT 

Bone Marrow Aspirate Kit

BMAK-01-RevD
2023-11-01

Paragon 28°, Inc. 
14445 Grasslands Dr.
Englewood, CO 80112 USA
(855) 786-2828

Paragon 28° Medical Devices Trading Limited
First Floor Block 7 Beckett Way
Park West Business Park
Dublin 12
D12 X884
Ireland
+353 (0) 1588 0350

Exclusively foot & ankle **28**
Paragon®

www.Paragon28.com

™Trademarks and *Registered Marks of Paragon 28°, Inc.
© Copyright 2023 Paragon 28°, Inc. All rights reserved.
Patents: www.paragon28.com/patents

For the contraindications, potential complications and adverse reactions, warnings and precautions associated with this device, please refer to the device specific instructions for use at <http://www.paragon28.com/ifus>