




## Gorilla® Calc Slide Plate

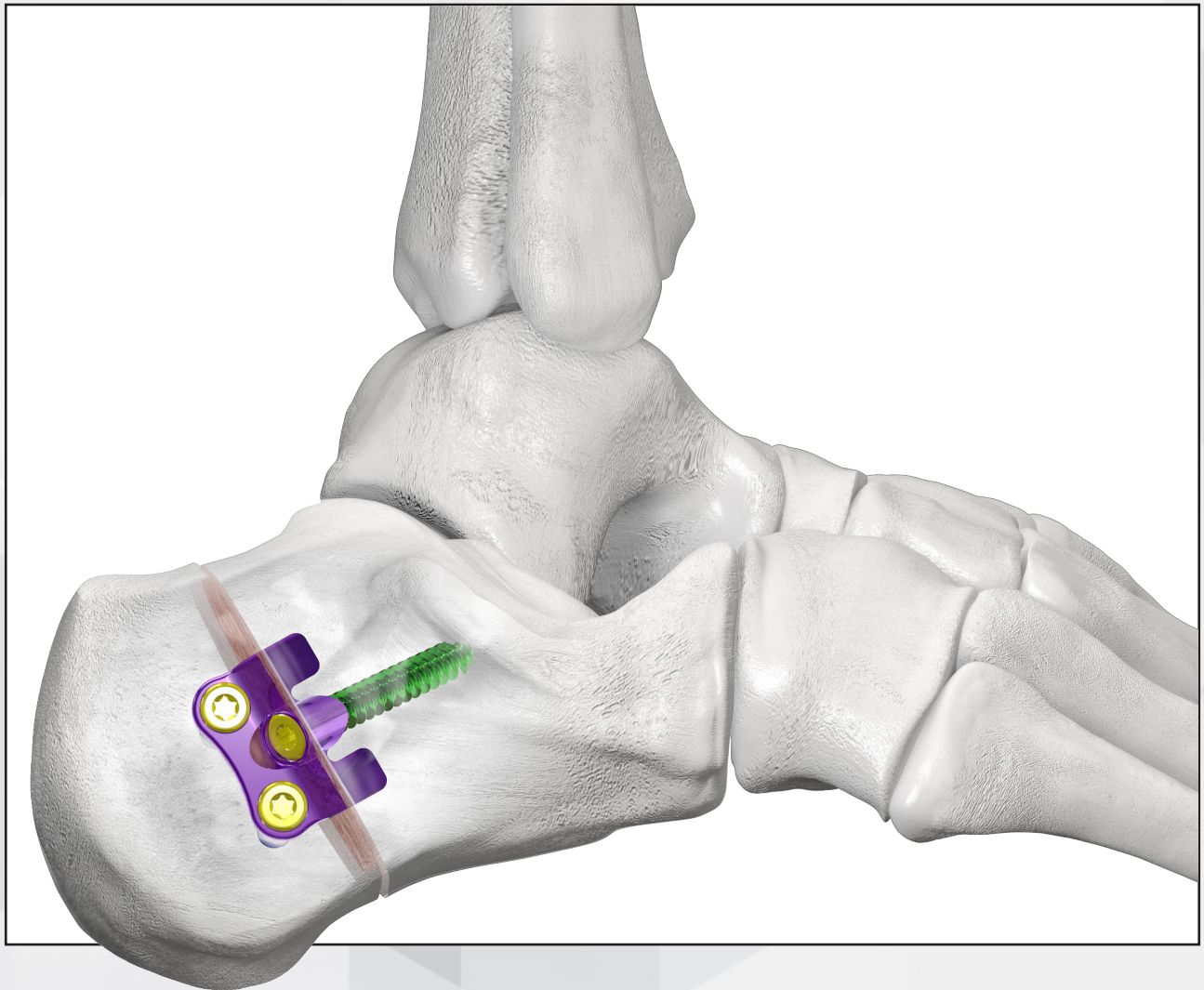

## CALC SLIDE PLATE


### System Highlights

- ▶ Intraosseous plating provides a firm buttress to prevent additional displacement of the osteotomy and avoidance of the peroneal tendons
- ▶ Straight forward technique—minimal fluoroscopy
- ▶ Plate is inserted through same incision as osteotomy
  - No violation of posterior heel pad with an incision or screw
  - May result in less hardware removal cases
- ▶ Plate hood allows for compression of posterior fragment, and includes angulation allowing the surgeon to capture the sustentaculum tali
- ▶ Does not violate growth plate of the calcaneus in pediatric patients


### Screw Options

All screw holes can accommodate 2.7 mm, 3.5 mm or 4.2 mm locking and non-locking variable angle plate screws

Length and width minimized to help prevent additional dissection and soft tissue disruption during plate insertion

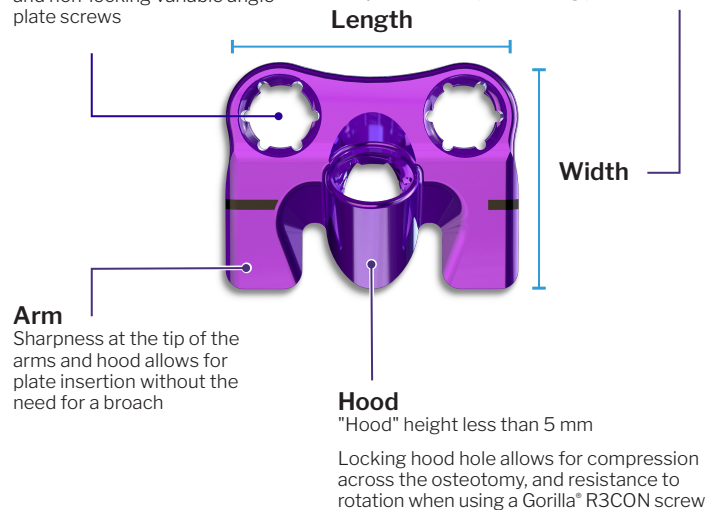

## INSERTER TOOL/SWIVEL GUIDE

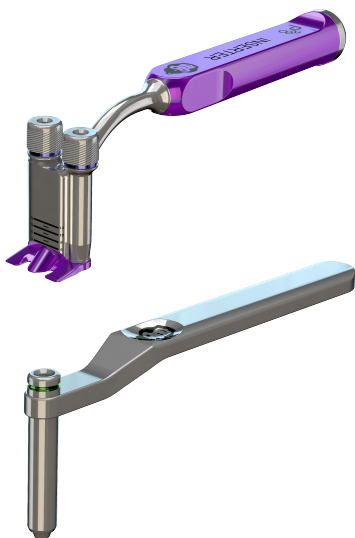

### Plate Inserter

- ▶ Connects to plate for easy insertion
- ▶ Laser marking on inserter measures osteotomy displacement
- ▶ Strike plate on tip of inserter allows insertion of plate with use of mallet
- ▶ Built in stop helps to prevent over-insertion of plate

### Swivel Guide

- ▶ Swivel guide assists in aiming cross screw toward sustentaculum tali
- ▶ Allows for off-axis drilling in any direction

## CALCANEAL SLIDE OSTEOTOMY

### Monster® Hindfoot Screw System

- ▶ Monster® Hindfoot Ø4.5 mm, Ø5.5 mm, and Ø7.0 mm screws available in headed and headless configurations
- ▶ Offered in 2 mm increments in all screw diameters for commonly used lengths
- ▶ Multiple thread lengths offered to maximize the amount of threads across the osteotomy
- ▶ Ø7.0 screw sizes offer Patent Pending Fluoroband K-wires which are notched at 20 mm and 32 mm to fluoroscopically determine optimal thread length


Short Threaded (16 mm)

Medium Threaded (20 mm)

Patent-Pending  
Fluoroband K-wire

Long Threaded (32 mm)

Fully Threaded

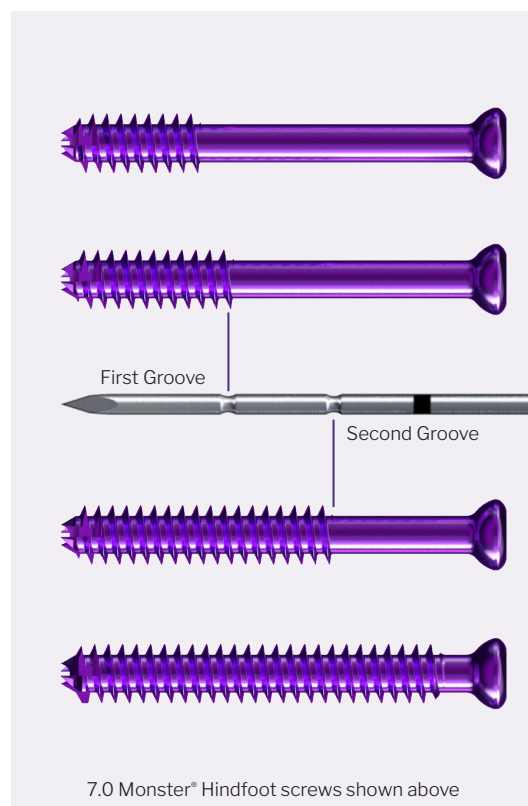

## OPTIONS FOR ANCILLARY PROCEDURES

HEvans® Evans Plate



Teddy Bear Plate



Dogbone Plate




NC Fusion Plate





## Gorilla® Calc Slide Plate

CSO-01-RevE  
2023-09-29

Paragon 28®, Inc.   
14445 Grasslands Dr.  
Englewood, CO 80112 USA  
(855) 786-2828

Paragon 28® Medical Devices Trading Limited  
First Floor Block 7 Beckett Way  
Park West Business Park  
Dublin 12  
D12 X884  
Ireland  
+353 (0) 1588 0350

™Trademarks and \*Registered Marks of Paragon 28®, Inc.  
© Copyright 2023 Paragon 28®, Inc. All rights reserved.  
Patents: [www.paragon28.com/patents](http://www.paragon28.com/patents)

Exclusively foot & ankle <sup>28</sup>  
**Paragon**®

[www.Paragon28.com](http://www.Paragon28.com)

For the contraindications, potential complications and adverse reactions, warnings and precautions associated with this device, please refer to the device specific instructions for use at <http://www.paragon28.com/ifus>