



PREPARATION GUIDE



Step 1

Open the pouch and remove the syringe. Confirm the syringe cap is securely locked.



Step 2

Place the syringe in the ambient sterile bath for 10–15 minutes, ensuring the contents are submerged.



Step 3

Remove the syringe from the sterile bath and unscrew the syringe cap from the top as shown above.



Step 4

Apply slight pressure to the plunger and extrude product out of the syringe into the palm of your hand.



Step 5

V92-FC+ is now ready to mold to desired shape. Product remains viable for up to 4 hours on the back table in an operative field.

P01-V92-PRFC+-RevC
2025-08-07

Paragon 28, Inc.
14445 Grasslands Dr.
Englewood, CO 80112 USA
(855) 786-2828



For the contraindications, potential complications and adverse reactions, warnings and precautions associated with this allograft, please refer to the allograft specific instructions for use at <http://www.paragon28.com/ifus>

Paragon 28® has used reasonable efforts to provide accurate information herein, but this information should not be construed as providing clinical advice, dictating reimbursement policy, or as a substitute for the judgment of a health care provider.

™ Trademarks and ® Registered Trademarks of Paragon 28, Inc. © Copyright 2025 Paragon 28, Inc.
All rights reserved.