



Optional Refrigerated Storage Expiration Record

Refrigerated Storage Starting Date*	
--	--

***ATTENTION: GRAFT EXPIRES TWELVE (12) MONTHS FROM DATE RECORDED.**

BioCushion[®] is preserved frozen and **must be maintained at a temperature of -40°C or colder** excluding the alternate refrigerated storage condition listed below:

- **Refrigerated storage between 0°C to 10°C is acceptable for a maximum of 12 months from refrigeration start date.** Once placed at refrigerated temperatures, BioCushion may NOT be refrozen, and the allograft must be used within twelve (12) months or discarded.
- If refrigerated storage is used, the facility must record refrigeration start date and establish a process to ensure the allograft is used within 12 months or discarded. Graft expires based on the recorded use-by-date, defined as twelve (12) months from the date recorded or the labeled expiration date, whichever comes first.

Product Code

Serial Number

"Use By" Date

Ensure this record is maintained with the allograft.