

Exclusively foot & ankle **20**
Paragon[®]

Bone Marrow Aspirate Kit

BONE MARROW ASPIRATE KIT

BONE MARROW HARVEST NEEDLE DESIGN

- Small 11 gauge needle designed to help minimize soft tissue disruption during retrieval of marrow at harvest site
- Easy insertion using a double diamond tip design for a faster harvest time

Multi-port fenestration to aid in bone marrow extraction

Double diamond tip for ease of insertion

Ergonomic handle to provide greater precision and leverage upon insertion


The Bone Marrow Aspirate kit is a great ancillary product to the PRESERVE™ Precut Allograft Wedge System

ALLOGRAFT FEATURES:


- Aseptically processed
 - No gamma irradiation or bleach
- Density matched
- Available for Evans, Cotton, MTP, Subtalar, Calcaneal Cuboid and Hammertoe procedures

LURE LOCK SYRINGE


Lure Lock syringe secures itself to the Harvest Needle construct to draw Bone Marrow continuously in an airtight environment

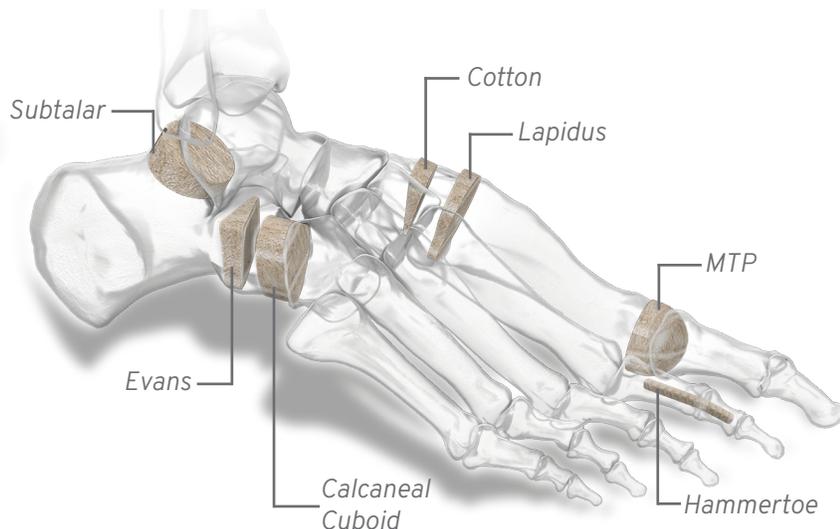


SURGICAL TECHNIQUE

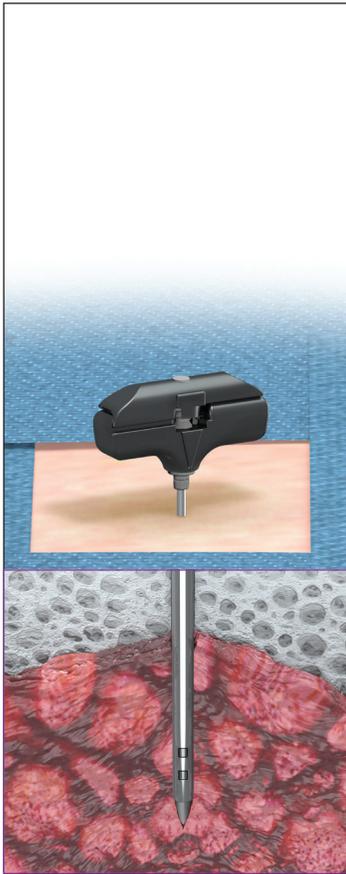


Make small incision at chosen harvest site.
Insert needle into harvest site.
Advance through cortex into marrow.

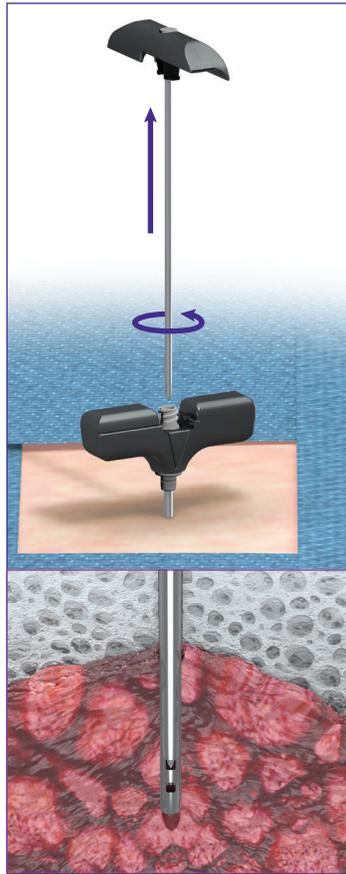

Twist top of handle 90° and remove needle from outer cannula.

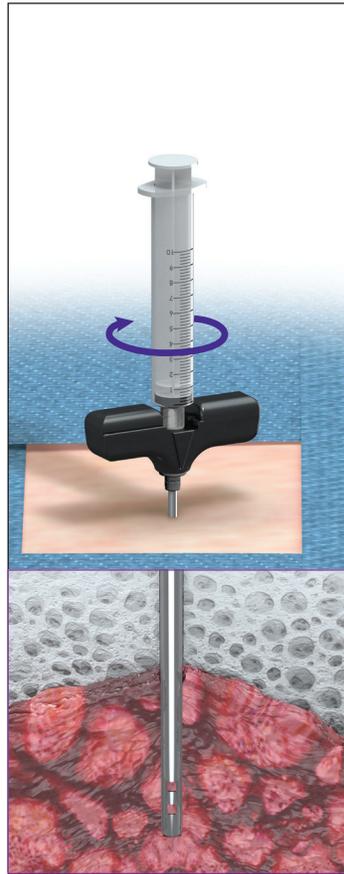

Secure lure lock syringe onto handle base.

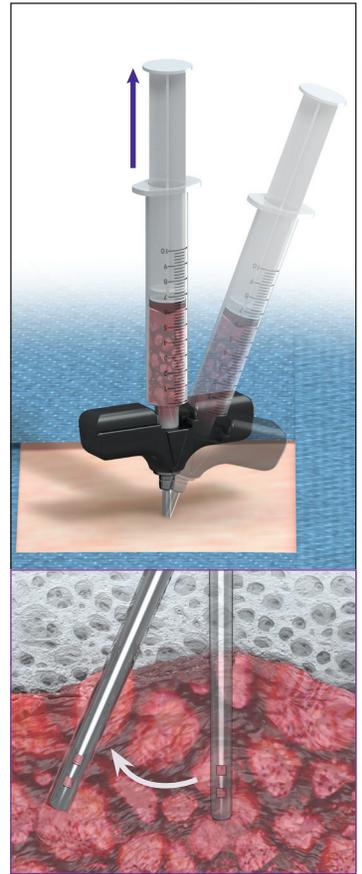

Extract bone marrow and re-position as needed to maximize bone marrow pull.

BONE MARROW ASPIRATE KIT CONTENTS

Description	Quantity
Bone Marrow Harvest Needle, 100 mm, 11 Gauge, Sterile	1*
Syringe, Locking, 10cc, Sterile	1*

* When ordered in the Bone Marrow Aspirate Kit, five individually packaged harvest needles and syringes will be provided; however, the facility will only be billed for the quantity used.

BMAK-01 RevC

™ Trademarks and ® Registered Trademarks of Paragon 28, Inc.
© Copyright 2020 Paragon 28, Inc. All rights reserved.
Patents: www.paragon28.com/patents

Paragon 28, Inc. 
14445 Grasslands Dr.
Englewood, CO 80112 USA
(855) 786-2828

Paragon 28 Medical Devices Trading Limited
First Floor Block 7 Beckett Way
Park West Business Park
Dublin 12
D12 X884
Ireland
+353 (0) 1588 0350



For additional information on Paragon 28® and its products please visit www.paragon28.com

