


Gorilla[®] Lateral Column Plating System

Features and Benefits:

- In the case where the cuboid is the cause of plantar ulcerations, anatomic reduction of the cuboid may help alleviate plantar pressure and aid in the healing of plantar ulcerations
- Plate and screw construct was designed to maintain anatomic alignment of the lateral column and prevent plantar subluxation of the cuboid
- The Gorilla[®] style Lateral Column Plate is designed to accept a Type II Anodized 5.5 mm beaming plate screw to help stabilize and compress the lateral column

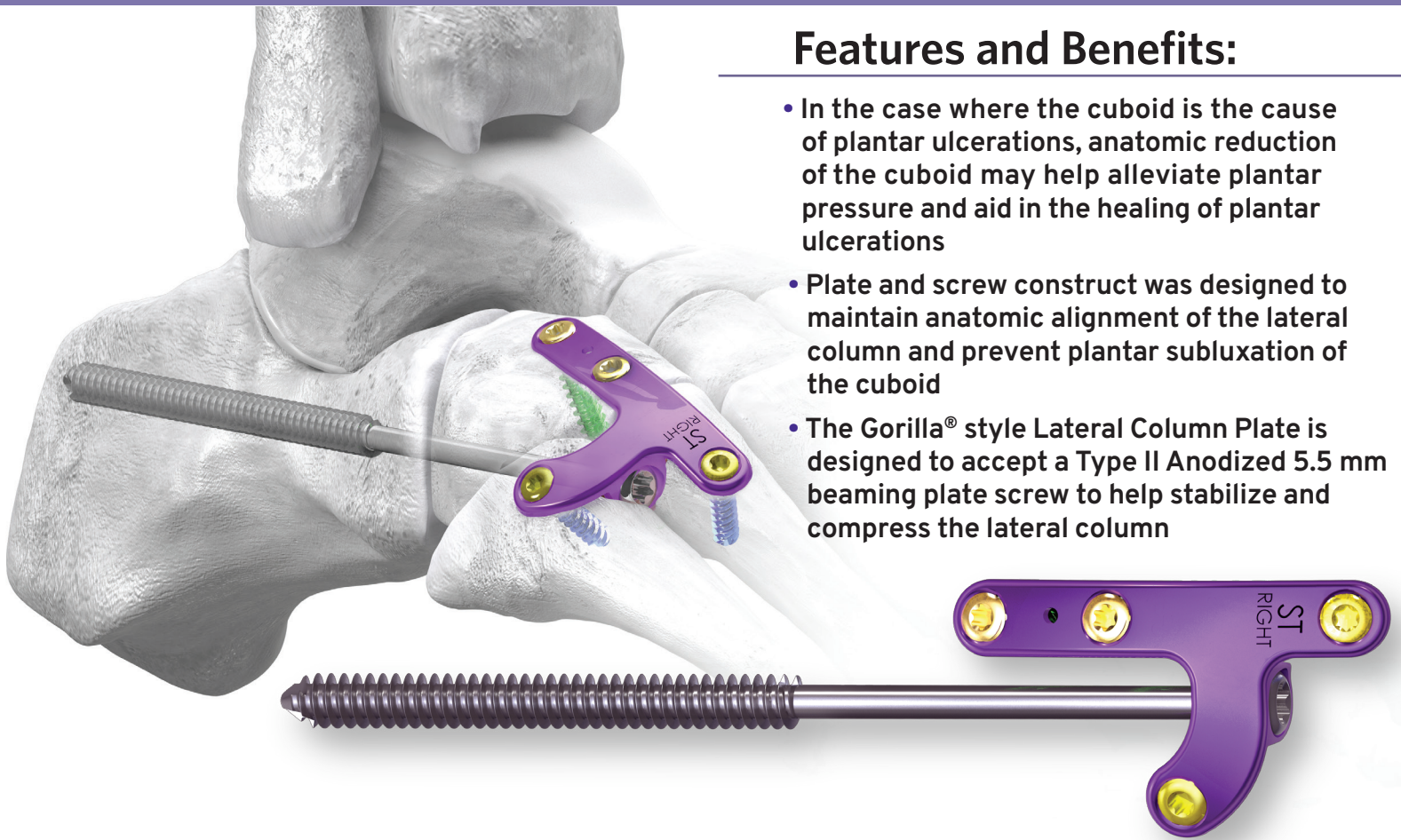

Lateral Column Plate

- Offered in Standard and Large sizes
- 1.5 mm plate thickness
- Anatomically contoured

Proximal screws on the cuboid help maintain anatomical alignment of the lateral column




Standard plate screw holes accept 2.7mm, 3.5mm and 4.2mm locking and non-locking variable angle plate screws

Recessed screw hole accepts a Type II Anodized 5.5 mm beaming plate screw

Type II Anodized 5.5 mm Beaming Plate Screw

- Offered in fully threaded and partially threaded options
- 5.5 mm screw interfaces with plate to help keep the cuboid from subluxing plantarly
- Offered in 80 mm - 120 mm lengths by 5 mm increments

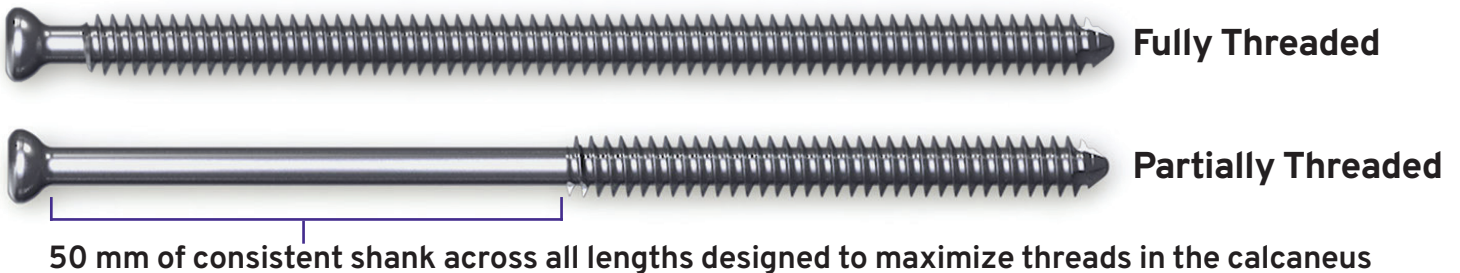

System Specific Instrumentation

Templating System



- Designed to establish correct reaming location between the 4th and 5th metatarsals to accommodate space for the 5.5 mm plate screw
- Joint indicating tab aligns template at the metatarsal-cuboid joint
- Wire holes through the template help facilitate easy transfer from template to implant



Joint Preparation Paddle

- Helps center the template in the inter-digital space so equal reaming of the 4th and 5th metatarsals can be achieved




Reamer

- Laser markings on reamer indicate appropriate reaming depth to accommodate recessed plate hole

LCFS-01 RevD

™ Trademarks and ® Registered Trademarks of Paragon 28, Inc.
© Copyright 2020 Paragon 28, Inc. All rights reserved.
Patents: www.paragon28.com/patents

Paragon 28, Inc. 
14445 Grasslands Dr.
Englewood, CO 80112 USA
(855) 786-2828

Paragon 28 Medical Devices Trading Limited
First Floor Block 7 Beckett Way
Park West Business Park
Dublin 12
D12 X884
Ireland
+353 (0) 1588 0350

CE 2797

For additional information on Paragon 28[®]
and its products please visit
www.paragon28.com

P28